



US FedEx Shipping Instructions:

- 1) Place the cardboard kit box containing the specimen and the completed test requisition form into the FedEx Clinical Pak.
- 2) Write your name and address in the space provided on the prepaid Billable Stamp and tear off the Customer Receipt for your records. Affix the Billable Stamp to the Clinical Pack in the space provided.
- 3) Call FedEx toll free at **1-800-238-5355** to schedule your pickup. When you hear the automated greeting say "REP". You will then be asked if you are calling to ship a package; reply "YES" to be connected directly to a live FedEx representative. Tell the representative that you need a pickup for a shipment using a prepaid "BILLABLE STAMP" and give your address.

Do not use a drop box. FedEx will not accept specimens placed into a drop box.

Doctor's Data provides prepaid shipping materials for use in shipping specimens from Canada, Ireland, United Kingdom, and the United States. If you are located outside of these countries or choose to use a different courier or level of service than provided, you must make your own shipping arrangements at your own expense.



US & CANADA: 1.800.323.2784 - UK: 0871.218.0052
ELSEWHERE: 1.630.377.8139 - FAX: 1.630.587.7860
3755 Illinois Avenue, St. Charles, IL USA 60174-2420
inquiries@doctorsdata.com - www.doctorsdata.com

Comprehensive Parasitology x3

Before You Start:

Please read all of the directions, and familiarize yourself with the collection procedures. The stool specimens for this test must be collected on three separate days. It is preferable, but not necessary, to collect on consecutive days but the final collection must be made within seven days of the first. If you are taking antifungal or antibiotic medications, please finish the course of medication, and then wait three days before starting this collection. Please refrain from taking digestive enzymes, antacids, and aspirin for two days prior to and during the specimen collection, unless otherwise instructed by your physician. *Never discontinue prescription medications without first consulting your physician.*

Verify Kit Contents:

- 1 Test requisition form
- 1 FedEx Clinical Pak with Billable Stamp
- 3 Collection containers (nested together; separate before use)

Day One Bag:

- 1 Black-capped vial
- 1 Disposable glove
- 1 Absorbent material

Day Two Bag:

- 1 Black-capped vial
- 1 Disposable glove
- 1 Absorbent material

Day Three Bag:

- 1 Black-capped vial
- 1 Yellow-capped vial
- 1 Disposable glove
- 1 Absorbent material

- ◆ If you are missing kit components, please call Doctor's Data's Customer Service department for assistance.
- ◆ Save the cardboard collection kit box to ship your samples back to the laboratory. Leave the absorbent material in the zip lock bag. Do not put it in the specimen vial.


Caution: Avoid contact of the skin and eyes with the fluid in the vials. For eye contact, flush with water thoroughly for 15 minutes. For skin contact, wash thoroughly with soap and water. For accidental ingestion, contact your physician immediately.

Collection One:

Use the contents of the Day One bag.




1) Collect your first stool specimen into the collection container. *DO NOT contaminate the specimen with either urine or water from the toilet.*

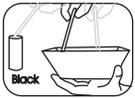

2) Unscrew the cap on the black-capped vial and, using the attached spoon, transport stool specimen into the vial. Take multiple portions from different areas of the collection container. Fill the vial to the fill line (approximately half full). **DO NOT OVERFILL.** Screw the cap on *tightly*.



3) Shake the black-capped vial vigorously for approximately 30 seconds to mix the stool specimen with the preservative in the vial.

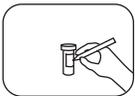

4) Write the patient's name, the date of collection, and patient's date of birth on the specimen vial. (You do not have to provide an ID#. The lab will assign one upon arrival). **The test cannot be performed without the patient information on the vial.**




5) Place the black-capped vial in the Day One zip-lock bag, then place the bag into the cardboard shipping box. Leave the box at room temperature.

Collection Two:

Use the contents of the Day Two bag.




6) Collect your second stool specimen into the collection container. *DO NOT contaminate the specimen with either urine or water from the toilet.*

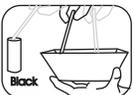

7) Unscrew the cap on the black-capped vial and, using the attached spoon, transport stool specimen into the vial. Take multiple portions from different areas of the collection container. Fill the vial to the fill line (approximately half full). **DO NOT OVERFILL.** Screw the cap on *tightly*.



8) Shake the black-capped vial vigorously for approximately 30 seconds to mix the stool specimens with the preservative in the vial.

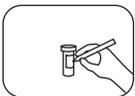

9) Write the patient's name, the date of collection, and patient's date of birth on the specimen vial. (You do not have to provide an ID#. The lab will assign one upon arrival). **The test cannot be performed without the patient information on the vial.**



10) Place the black-capped vial into the Day Two zip-lock bag, then place the bag into the cardboard shipping box. Leave the box at room temperature.

Collection Three:


Use the contents of the Day Three bag.




11) Collect your third stool specimen into the collection container. Remember not to contaminate the specimen with urine or water from the toilet.



12) Unscrew the cap on the yellow-capped vial and, using the attached spoon, transport stool specimen into the vial. Take multiple portions from different areas of the collection container. Fill the vial to the fill line (approximately half full). **DO NOT OVERFILL.** Screw the cap on tightly.

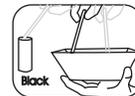

13) Unscrew the cap on the black-capped vial and, using the attached spoon, transport stool specimen into the vial. Take multiple portions from different areas of the collection container. Fill the vial to the fill line (approximately half full). **DO NOT OVERFILL.** Screw the cap on tightly.

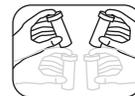

14) Shake both the yellow and black-capped vials vigorously for approximately 30 seconds to mix the stool specimens with the preservatives in the vials.



15) Write the patient's name, the date of collection, and patient's date of birth on both specimen vials. (You do not have to provide an ID#. The lab will assign one upon arrival). **The test cannot be performed without the patient information on the vials.**



16) Place both the yellow and black-capped vials into the Day Three zip-lock bag, then place the bag into the cardboard shipping box. Check to be sure that all of the bags and vials are in the box.



17) Fill out the test requisition form completely and sign it. **The test cannot be performed without the patient information on the vial and a properly filled out requisition.** Place the form in the cardboard shipping box. The specimens are now ready for shipment.

To obtain the best results, Doctor's Data recommends shipping specimens as soon as possible. If you cannot ship the specimens the same day you finish the collection, such as over a weekend or holiday, place the cardboard box containing the yellow and black-capped vials into a refrigerator until you are ready to ship the specimens.

Consult your physician if you have any questions during the test collection process.