



US FedEx Shipping Instructions:

1) Place the cardboard kit box containing the specimen and the completed test requisition form into the FedEx Clinical Pak.

2) Write your name and address in the space provided on the prepaid Billable Stamp and tear off the Customer Receipt for your records. Affix the Billable Stamp to the Clinical Pack in the space provided.

3) Call FedEx toll free at **1-800-238-5355** to schedule your pickup. When you hear the automated greeting say "REP". You will then be asked if you are calling to ship a package; reply "YES" to be connected directly to a live FedEx representative. Tell the representative that you need a pickup for a shipment using a prepaid "BILLABLE STAMP" and give your address.

Do not use a drop box. FedEx will not accept specimens placed into a drop box.

Doctor's Data provides prepaid shipping materials for use in shipping specimens from Canada, Ireland, United Kingdom, and the United States. If you are located outside of these countries or choose to use a different courier or level of service than provided, you must make your own shipping arrangements at your own expense.



US & CANADA: 1.800.323.2784 - UK: 0871.218.0052
ELSEWHERE: 1.630.377.8139 - FAX: 1.630.587.7860
3755 Illinois Avenue, St. Charles, IL USA 60174-2420
inquiries@doctorsdata.com - www.doctorsdata.com

Collection Instructions for the following tests: **Toxigenic C. Difficile DNA, Calprotectin, Elastase, H. Pylori, Lactoferrin, Lysozyme, Secretory IgA**

Before You Start:

Please read all of the directions, and familiarize yourself with the collection procedures. If you are taking antifungal or antibiotic medications, please finish the course of medication, and then wait three days before starting this collection. Please refrain from taking digestive enzymes, antacids, and aspirin for two days prior to and during the specimen collection, unless otherwise instructed by your physician. *Never discontinue prescription medications without first consulting your physician.*

Verify Kit Contents:

- 1 Test requisition form
- 1 Polar Pack gel pack
- 1 Collection container
- 1 White-capped vial
- 1 Disposable glove
- 1 Zip-lock bag with absorbent material
- 1 FedEx Clinical Pak with Prepaid Billable Stamp

- ◆ If you are missing kit components, please call Doctor's Data's Customer Service department for assistance.
- ◆ Save the cardboard collection kit box to ship your samples back to the laboratory. Leave the absorbent material in the zip lock bag. Do not put it in the specimen vial.

